



**ALTOMED**

2 Witney Way  
Baldon Business Park  
Tyne and Wear  
NE35 9PE  
England  
Tel: 0191 519 0111  
Fax: 0191 519 0283

## **The Crawford Fascia Lata Stripper and Separator**

The following products are now available from Altomed Limited.

**To place an order contact Customer Services on  
Tel: (0) 191 519 0111 Fax: (0) 191 519 0283**

### **A6368 - Crawford Fascia Stripper**

#### **Product Description:**

Overall length approximately 355mm.



#### Instructions for Crawford Fascia Stripper

To release for operation, turn off-centre disc next to finger rings one-half turn. Cutting blade is then free to move for use.

To disassemble for cleaning, swing right finger ring out, releasing the channel clip, and pull blade straight back, lifting the cutting blade out of its operating groove.

After cleaning, sterilize in disassembled state.

To replace, return cutting blade into its channel, dropping the two pivot posts into the appropriate notches. Push blade forward and swing ring clip over channel.

### **28-0188 Crawford Fascia Separator**

This instrument has been designed to help separate and obtain fascia lata. It is of greatest use with children 3½ years and older. See attached document "*A New Instrument For Separating Fascia Lata in young Children.*"